Advancis

Medical

Activon[®]

Tube

Instructions

for use

EN Instructions For Use

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Activon® Tube

100% Medical Grade Manuka honey

Product description

Activon® Tube is 100% Medical Grade Manuka honey which has a reliable level of antibacterial potency not found in other honeys. In addition to its antibacterial properties, this wound filler benefits patients with wounds in a number of ways: it has an osmotic effect that helps to debride and deslough wounds, reduces odour and maintains a moist healing environment.

Indications

For use on Activon® honey impregnated wound dressings or as a wound filler, directly onto the wound bed, for partial or full thickness wounds including; sloughy wounds, pressure ulcers, surgical wounds, burns, graft sites and malodorous wounds.

Contra-Indications

Known allergy to bee-venom/honey.

Directions for use

Ensure the wound has been cleaned, gently dry the surrounding skin. Remove the sterile tube from the retail box. Twist off the Activon® Tube cap, apply liberal amounts of the Activon® Tube to the selected dressing of choice or as a filler directly onto the wound bed. Apply the selected secondary dressing as per the 'Instructions for Use' supplied with the dressing. Single patient use only.

Can be used for up to 90 days after opening.

Warnings and

Precautions

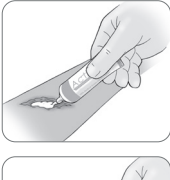
Although no known instances of increased levels of blood sugar in patients with diabetes, it is advisable to closely monitor the levels as this product contains glucose. A few patients may experience pain when applied, depending on the sensitivity of the wound, it may be necessary to consider an appropriate level of analgesia. If pain continues unabated, discontinue use and irrigate the wound with a saline solution.

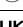
Storage

Store at room temperature. Our honey is 100% Manuka honey. As a natural product this can vary in colour and in exceptional circumstances crystallisation can occur.

Disposal

Dispose of dressing as 'Clinical Waste'

	Catalogue Number
	Do not re-use
UK CA	UKCA mark for Great Britain
	Caution, consult instructions for use
	Quantity
	Unique Device Identifier (UDI carrier including the AIDC and human readable information)
	Date of manufacture/ Country of manufacture
	The product has made a financial contribution towards the recovery and recycling of packaging in Europe
	Consult instructions for use
	Keep dry
	Keep away from sunlight
	Manufacturer
	Authorized representative in the European Community
	Batch code
	CE Mark for Northern Ireland, EC & EU
	Do not re-sterilize
	Do not use if package is damaged
	Importer
	Single patient multiple use
	Sterilized using irradiation
	Quantity

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Brightwake Ltd,
Ltd,